



derma+flex[®]

TOPICAL SKIN ADHESIVE
High Viscosity Medical Cyanoacrylate

Easy Application. Effective Formula.

How to Use derma+flex[®]



1

Screw the Applicator onto the threaded tube in a clockwise direction until the applicator is tight and seated on the tube. The applicator will puncture the foil membrane on the tube allowing adhesive to flow.



2

While holding the tube upright and away from patient, squeeze the tube from the bottom moving the adhesive upward until it becomes visible on the applicator surface and is flowing freely.



3

While holding the flattened part of the tube, begin "painting" the area of the skin that requires adhesive in a back and forth motion, covering the entire surface of the approximated skin edges as well as the area around the edges.



4

Continue to hold the wound edges in approximation for 60 seconds until the adhesive cures.



5

For fine line application simply tilt the tip to use the fine edge of the applicator as picture 5, for wide lines tilt to use the flat side of the applicator as picture 3

